

Bi-Flow Femoral Cannulae – Important Safety Information

203-XXX

INDICATIONS:

BI-FLOW is a single-use STERILE device intended as a femoral perfusion cannula to return arterial blood from the extracorporeal circuit to the patient's systemic circulation and lower limb during cardiopulmonary surgery for periods up to 6 hours.

CONTRAINDICATIONS:

The cannulae are for use only for the above-mentioned purpose. Not recommended for use in diseased or abnormal cannulation sites.

GENERAL WARNINGS:

1. The User should carefully check the device during set-up and priming for leaks. Do not use if any leak is detected.
2. The device must be used in accordance with the instructions for use provided in this manual.
3. For use by professionally trained personnel only. Sorin Group Italia is not responsible for problems arising from inexperience or improper use.
4. Proper surgical procedure and technique is the responsibility of the attending medical professional (physician). This guideline is furnished for informational purposes only. Each surgeon must evaluate the appropriate use of this device, case by case, based upon medical training, experience, and the type of surgical procedure employed.
5. FRAGILE, handle with care.
6. Keep dry. Store at room temperature.
7. For single use and for single-patient use only. During use the device is in contact with human blood, body fluids, liquids or gases for the purpose of eventual infusion, administration or introduction into the body, and due to its specific design it cannot be fully cleaned and disinfected after use. Therefore, reuse on other patients might cause cross-contamination, infection and sepsis. In addition, the reuse increases the probability of product failure (integrity, functionality and clinical effectiveness).
8. Sterile Contents/Non-Pyrogenic Fluid Pathway unless package is opened or damaged. Do not use if sterile packaging is damaged, unsealed, or has been exposed to moisture or other conditions that would compromise the sterility of the device.
9. The device must not undergo any further processing.
10. Do not resterilize.
11. After use, dispose of the device in accordance with applicable regulations in force in the country of use.
12. Check the expiration date on the label attached. Do not use the device after the date shown.
13. The device must be used immediately after opening the sterile packaging.
14. The device must be handled aseptically.
15. Do not use solvents such as alcohol, ether, acetone, etc.: as contact may cause damage to the device.
16. Do not cut or alter the cannula in any way. Cutting the cannula could release particulate matter which may be thrombogenic or embolic and may render the device unusable.



Health innovation that matters

For professional use. Please visit www.sorinmanuals.com to find instructions for use containing full prescribing information, including indications, contraindications, warnings, precautions and adverse events. Consult your labeling.

Manufacturer:
Sorin Group Italia S.r.l.
Strada Statale 12 Nord, 86
Mirandola (MO) 41037
Italy