



Health innovation that matters

## **Dilators' Kit – Important Safety Information**

200-120

### **INDICATIONS:**

The Vascular Dilator Kit is intended for use in surgical procedures to aid in vascular insertion of a catheter or cannula.

### **CONTRAINDICATIONS:**

The guidewire is contraindicated for use in the coronary or cerebral vasculature.

### **GENERAL WARNINGS:**

- This device is only intended to be used by professionally trained personnel. Proper surgical procedures and techniques are necessarily the responsibility of the medical professional. These instructions are furnished for informational purposes only. Each surgeon must evaluate the appropriate use of this device, case by case, based on medical training, experience, and the type of surgical procedure employed.
- For single use and for single patient use only. During use, the device is in contact with human blood, body fluids, liquids or gases for the purpose of eventual infusion, administration or introduction into the body and due to its specific design, it cannot be fully cleaned and disinfected at the end of use. Therefore, reuse on other patients might cause cross-contamination, infection and sepsis. In addition, reuse increases the probability of product failure (integrity, functionality and clinical effectiveness).
- Do not use this device if package is damaged or opened.
- Never use a device if it is beyond its expiration date.
- Do not use this device if it shows signs of damage, e.g., crimps, kinks, or crushed areas.
- Handle with care after use as the product may be contaminated with blood and/or body fluids. Dispose of in accordance with applicable regulations in the country of use.
- DO NOT REUSE. DO NOT RESTERILIZE.

For professional use. Please visit [www.sorinmanuals.com](http://www.sorinmanuals.com) to find instructions for use containing full prescribing information, including indications, contraindications, warnings, precautions and adverse events. Consult your labeling.

Manufacturer:  
LivaNova USA Inc.  
14401 West 65<sup>th</sup> Way,  
Arvada CO 80004,  
United States